

# Protocole de forage - **LIKE BS**



Diamètre implant	Séquence de base	Densité osseuse	Séquence de forage
<b>NP</b> Ø 3,3 mm	<b>1</b>	Soft D4-D3	--
		Medium D3-D2	⊕ <b>2</b>
		Hard D2-D1	⊕ <b>2 3</b>
<b>RP</b> Ø 3,75 mm	<b>1 2 3</b>	Soft D4-D3	--
		Medium D3-D2	⊕ <b>4</b>
		Hard D2-D1	⊕ <b>4 5</b>
<b>RP</b> Ø 4 mm	<b>1 2 3 4</b>	Soft D4-D3	--
		Medium D3-D2	⊕ <b>5</b>
		Hard D2-D1	⊕ <b>6</b>
<b>WP</b> Ø 5 mm	<b>1 2 3 4 5</b>	Soft D4-D3	--
		Medium D3-D2	⊕ <b>6</b>
		Hard D2-D1	⊕ <b>6 7</b>